

How to connect the Orbit Reader to an Android device

Install BrailleBack on your Android device. This software can be downloaded via the Play Store on your device.

Then, turn on BrailleBack and pair your braille display. These steps are explained below.

Note: Before you turn on BrailleBack, make sure that TalkBack is also on.

1. Open your device's Settings app
2. Open Accessibility, then BrailleBack
3. Turn on BrailleBack

On your Orbit Reader 20, press Space + Dots 4 7. Bluetooth Mode should be displayed.

1. On your phone, navigate to Settings. Under Wireless & Networks, select Bluetooth and then turn on Bluetooth.
2. In the list that appears, select the name of your braille display which would be Orbit Reader 20 and the serial number of your device.

In a few seconds the connection should happen and you will find the braille display updating as you navigate around the screen on your phone.

BrailleBack Commands

Function	Keys
Back	Space + B
Home	Space + H
BrailleBack help	Space + Dots 1 2 3
Notifications	Space + N
Recent apps	Space + R
Enter (when editing text)	Space + Dot 8
Backspace (when editing text)	Space + Dot 7
Navigate to the right or left	Right or Left Pan Key
Switch between literary and computer braille	Space + G
Search the screen for text	Space + Dots 3 4