

How to read files on the Orbit Reader

When you turn the device on, Orbit Reader 20 resumes from the last place of the last file in use. To get to the file list, press Dot 7. Go to the desired file using Up or Down Arrow key. Press Dot 8 or Select to open the file.

- Go to Previous or Next display: Panning keys
- Go to Previous or Next Character: Space + Dot 3 or Space + Dot 6
- Go to Previous or Next Word: Space + Dot 2 or Space + Dot 5
- Go to Previous or Next Paragraph: Space + Dot 1 or Space + Dot 4
- Go to Top or Bottom of a file: Space + Dots 1 2 3 or Space + Dots 4 5 6

There are many other functions in reading mode such as Find, Power Move, Bookmarks, and more.